


ANK - Anklet

EN	ANKLE/FOOT ORTHOSIS
DE	FUßGELENK-/FUßORTHESE
FR	ORTHÈSE DE CHEVILLE/PIED
ES	ORTESIS DE TOBILLO/PIE
IT	ORTESI DI CAVIGLIA/PIEDE
NL	ENKEL-/VOETORTHESE
DK	ANKEL-/FODORTOSE
FI	NILKKA-JALKATERÄORTOOSI
SE	ORTOS, FOT/FOTLED
NO	ORTOSE, ANKEL/FOT
PL	ORTEZA KOSTKI/STOPY
GR	ΝΑΡΘΗΚΑΣ ΑΣΤΡΑΓΑΛΟΥ/ΠΟΔΙΟΥ

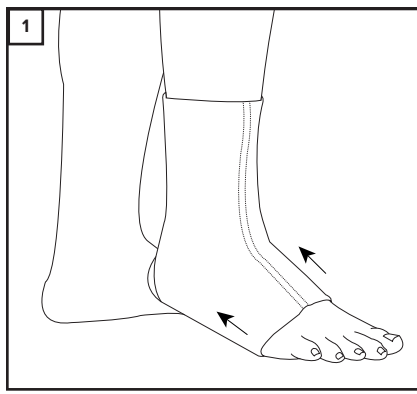

ENGLISH

Product Description

Ankle compression support brace.

Usage Information (See Image)

Position brace on ankle as shown.

Indications for Use

- Instability in the ankle.

Contraindications

- Do not apply in direct contact with broken skin.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Polyester, SBR.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

MD MEDICAL DEVICE

1 SINGLE PATIENT - MULTIPLE USE

! CAUTION



Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

CE **EC** **REP** **EMERGO EUROPE**
Prinsessegracht 20
2514 AP The Hague
The Netherlands